


Marpac[®]

Neo/Ped ET Tape, 308 series Instructions

1. Clean and dry the patient's face

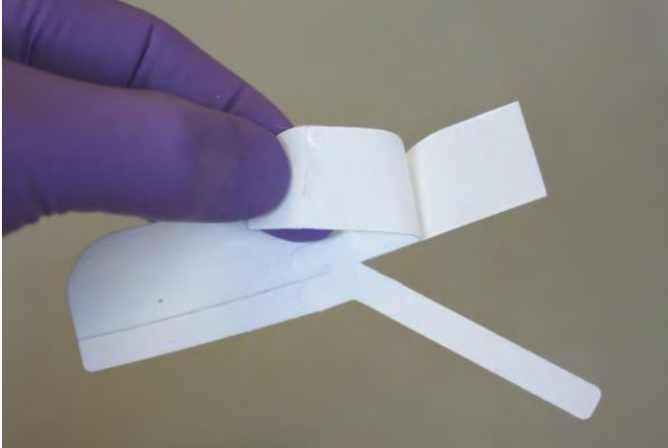

2. Bend the tape to expose the break in the paper backing and remove the largest section.



3. Center and apply to the patient's face. Remove the paper backing from the angled leg first.



4. Wrap the angled leg up the tube, **leaving space between each wrap**. Wrap the opposing leg.



5. Apply the support strip 1/4" away from lip and under the tube, then back on to the patient's face.



6. Apply the cover piece over the entire product. This provides more support for the support strip.



7. The finished product applied to the patient.